




Surgical Style Face Mask Instructions by beadnova.com

Step 1: Cut the fabric

Cut the fabric with your size. You can place our pattern on your fabric and follow the pattern to cut. (Be noted that the size of our pattern is half of actual size, fold the fabric first when cutting the fabric.)

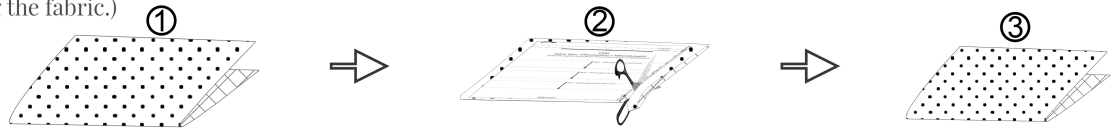


Step 2: Sew the short side with 10mm seam allowance

Fold the short side of fabric toward wrong side and sew the edge with 10mm seam allowance.




Step 3: Fold the fabric to half and sew to create a filter pocket

At this step, fold the fabric in half with right side facing each other. Then sew the edge of long side with 5mm seam allowance but retain the middle 100mm (you can use sewing pins to pin the position) to create a filter pocket. Then move the open pocket from top to the middle.

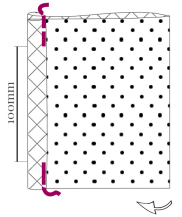


Step 4: Add elastic bands

At this step, the edge of left and right sides is opened, put two elastic bands in the end of both sides separately, and then sew the edge with 5mm seam allowance to fix the elastic band on fabric.

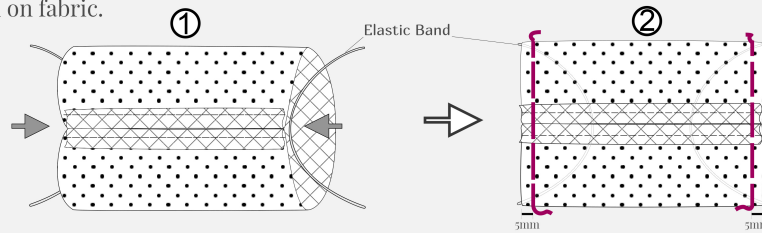


Step 5: Turn the right side of fabric out from the 100mm open pocket

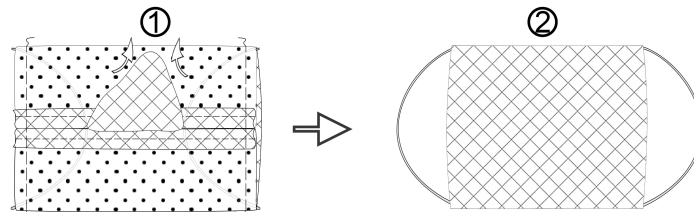

Step 6: Add wire for nose bar

Back of mask facing you and the middle is the open pocket. Insert wire at top inside the pocket and sew from left to right to fix the position of wire.

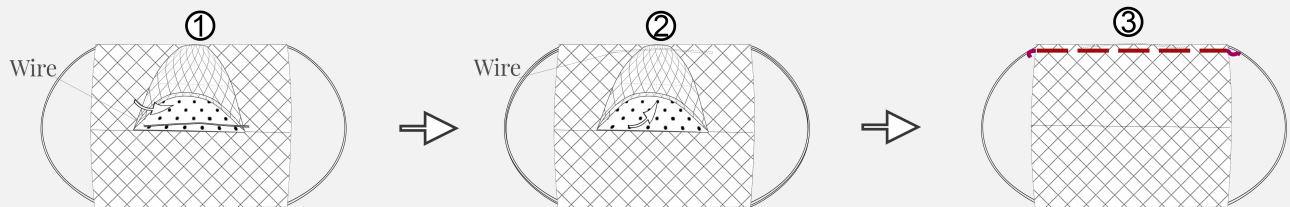

Step 7: Make 3 pleats of fabric

Turn the mask that front side facing you, place our folding template on fabric and fold 3 pleats (or use ruler and pen to mark down the position of 3-pleat on the fabric, then follow the position you marked and fold the 3 pleats), then use sewing pins to fix it and sew the edge.

